
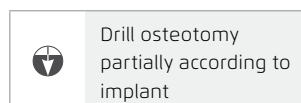
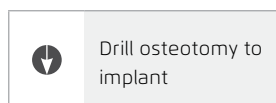


ONE-PIECE SERIES | MBI NC™

RECOMMENDED STRAIGHT DRILL PROTOCOL

IMPLANT DIAMETER	Drill Diameter [mm]	Ø1.2	Ø1.5	Ø2.0
	Drill Speed [RPM]	1200-1500	1200-1500	900-1200
	Ø2.0	Soft Bone 2/3	Hard Bone	
	Ø2.4	Soft Bone 2/3	Hard Bone	
Ø2.9	Soft Bone			
	Hard Bone	 2/3	

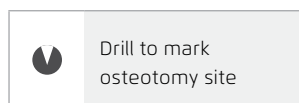

The recommended drill protocol procedure should not replace the dentist's/surgeon's judgment. The implants may be loaded for immediate function when good primary stability (above 35 Ncm) has been achieved and with appropriate occlusal loading.

MBI & MBI NC COMPONENTS

