


CHALLENGE SERIES | ZYGOMATIC™

DRILLING SEQUENCE FOR ZYGOMATIC

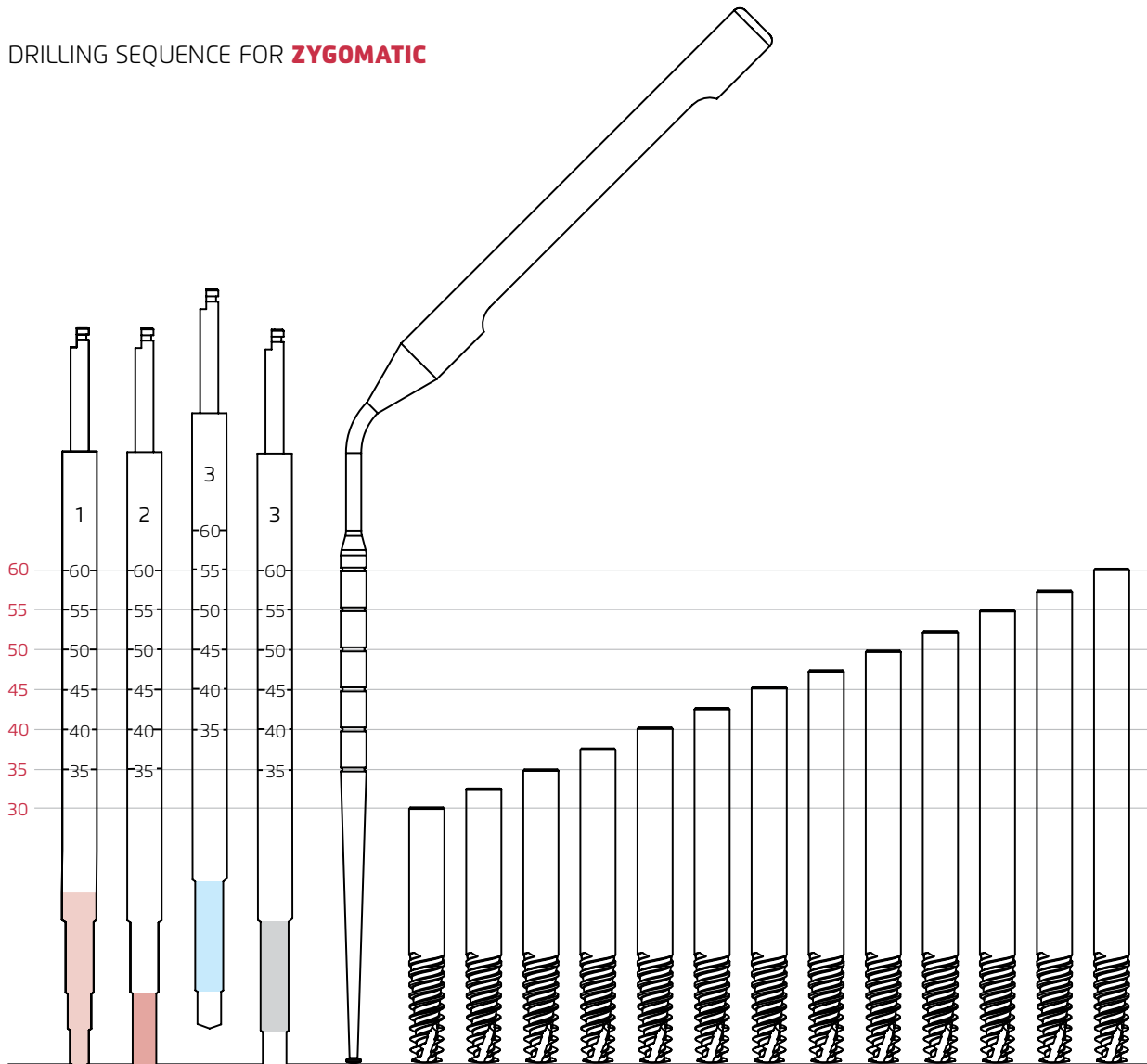

RECOMMENDED STEP DRILL PROTOCOL

Drill Diameter [mm]		1	2	3
Drill Speed [RPM]				
IMPLANT DIAMETER	Bone Type			
	D4	↓		
	D3	↓	→ ↓	
	D2	↓	→ ↓	→ ↓ -5mm
	D1	↓	→ ↓	→ ↓

↓	Drill osteotomy to implant	-5mm	Drill 5mm Less
---	----------------------------	------	----------------

Prior to the use of Zygomatic implants in the Zygoma region, additional training is recommended.